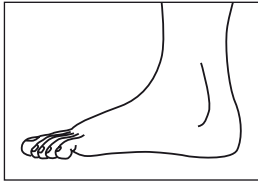
# GIBAUD

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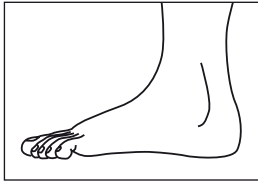
Measurement exact:  
shoe size.

## ARTICULATED MALLÉOGIB® 2

The brace will be issued and used based on a prescription.

If you have any doubts regarding the application and use, do not hesitate to ask your doctor and/or fitter, pharmacist, orthotist or orthopaedic specialist for advice.

**Do NOT drive a vehicle while wearing this brace.**

### INDICATION(S):

- Recent moderate or severe ankle sprain,
- Rehabilitation phase, return to sport,
- Functional instability, ligament hyperlaxity,
- Recurrent sprain, chronic laxity.

### PRODUCT ADVANTAGES:

- Ankle stabilisation in the frontal plane,
- Optimal stabilisation through the combination of a heel cup to hold the rearfoot and two rigid shells,
- Articulation that reproduces physiological movements,
- Option to lock the joint straight during the acute phase,
- Supple (wrap around), ventilated padding to help reduce oedema,
- Low-bulk brace that is comfortable to wear.

### FITTING:

- In addition to measuring the foot, try on the brace to ensure a good fit and to understand how to put it on properly.
- Then put on the brace in the same way as it was first put on by the health professional.

- 1- Choose the size that corresponds to your shoe size.
- 2- Open the two straps and spread apart the shells.
- 3- Place the heel into the opening on the cup.
- 4- Apply the shells on each side of the foot so that the malleoli are covered.
- 5- Tighten the lower strap and then the upper strap.
- 6- With the brace on, slide the foot into a spacious, stable shoe. Do not use pumps or any shoes with high heels. If the shoe has a thick, removable inner sole, taking it out will make walking more comfortable.

- 7- Loosening may occur with use. Retighten the straps if this occurs.
- 8- If you feel local discomfort and/or tingling, release the straps and reposition the shells.
- 9- On medical advice, the joint on the brace can be temporarily locked using the screws provided in the box.

**Note: The joint can be temporarily locked according to your doctor's recommendations to manage the oedema (first six days).**

### CONTRAINDICATIONS AND LIMITATIONS OF USE:

- Serious injury requiring medical or surgical treatment.

### PRECAUTIONS FOR USE:

- Although walking is more secure while wearing the brace, avoid prolonged ambulation to avoid aggravating the local signs.
- You must wear either street shoes or sports shoes while walking.
- Avoid moving around with the brace while not wearing shoes on slippery floors, especially if the floor is wet (shower room).
- Do not place the brace in direct contact with any skin lesions.

### SIZE:

The brace can be used either on the left or right ankle.

Shoe size	34 - 37	38 - 42	43 - 46
Size	1	2	3

### HEALTH EDUCATION ADVICE:

- Although this brace is removable, initially it is advised to wear it day and night until the next visit.
- Even if your pain is gone, do not stop wearing the brace **unless instructed to do so by your doctor.**
- To limit the risk of complications, your doctor may advise you to
  - Avoid standing and not moving,
  - Elevate your leg whenever possible,
  - Apply ice to the outside of your ankle (ice container surrounded by a cloth) for 20 minutes, three to four times per day.
- In case of pain and/or significant local signs (haematoma, oedema), your doctor may suggest wearing the brace but not putting weight on your foot and using forearm crutches for a few days.

### CARE AND MAINTENANCE:

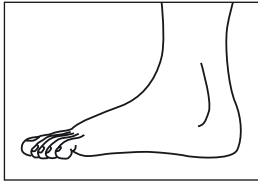
Wipe the plastic parts with a clean, damp cloth. The removable lateral protection can be machine washed (delicate cycle, low temperature).

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